

## INSTRUCTIONS FOR USE Conti Spray (ECG Spray)

ContiSpray is a very good electrotransmission gel for ECG diagnostics, designed to reduce the resistance between skin and electrodes. ContiSpray is watersoluble, drying-resistant, easily removable. ContiSpray doesn't stain, does not grease. It does not contain any active principles that might damage the electrodes.

### INSTRUCTIONS FOR USE

- Spray is ready to use
- Clean and degrease the patient skin.
- Apply the desired amount of Spray.
- After the procedure remove all residues of the spray with paper, towel or gauze moistened with water.
- Stop using the gel for hypersensitive patients, if skin reactions occur.

### WARNINGS

- External use only.
- The product is not sterile.
- Do not apply spray on skin affected by abrasions, erythema or wounds.
- Close the bottle after use.
- May cause irritations or reddenings with allergic persons.
- Contact with eyes may cause irritation.
- Analyst with sensitive skin should wear protective gloves during the use.
- Do not use the spray after the "use by" date on the bottle.
- If the package contains a labeled, empty refillable bottle, only the product with the name on the label should be filled

**INGREDIENTS:** aqua, propylene glycol, sodium chloride, phenoxyethanol, ethylhexylglycerin E131, E 102

### PACKAGING

250 ml (35 / box)

5000ml (4 / box)

### DISPOSAL CONSIDERATION

Product, waste: Chemicals must be disposed of in compliance with the respective national regulations. Packaging must be disposed in compliance with the country-specific.













### STORAGE

Temperature: +10°C ÷ +30°C - Relative humidity: 30% ÷ 75%. Keep container firmly closed, away from direct sunlight and sources of UV radiation.

**EXPIRY:** 36 months from production date. Use only 3 month after opening.

**NOTICE:** any adverse event that has occurred in relation to the device should be reported to the manufacturer and any serious incident (MDR Article 2 (65)) should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

02.05.2022 V01

	Manufacturer		Temperature limit		Batch-Code		Attention
	User Manual		polyethylene packaging		Use by date		Do not reuse
	no steril		away from UV		Medical device		

### Ultragel Medical Kft.

✉ 1022 Budapest, Aranka utca 12.

☎ +36 1 278 3050

✉ [info@ultragelmedical.hu](mailto:info@ultragelmedical.hu)

🌐 [www.ultragelmedical.hu](http://www.ultragelmedical.hu)