



**INSTRUCTIONS FOR USE**  
**RF cream „premium„**

RF cream „ premium „ is a viscous creamy gel.A conductive gel created for physiotherapy medicine treatments that use electrical energy and radiofrequency (RF) to boost their effectiveness. It contribute the suitable connection between electrodes, probes, and skin.

**INSTRUCTIONS FOR USE**

- \* Gel is ready to use
- \* Clean and degrease the patient's skin.
- \* Apply the desired amount of gel
- \* Connect and use the equipment by following the instructions supplied by the manufacturers.
- After the procedure remove all residues of the gel with paper towel or gauze moistened with water.
- \* Stop using the gel for hypersensitive patients, if skin reactions occur.

**WARNINGS**

- External use only.
- The product is not sterile.
- Do not apply gel on skin affected by abrasions, erythema or wounds.
- Close the bottle after use.
- May cause irritations or reddenings with allergic persons.
- Contact with eyes may cause irritation.
- Analyst with sensitive skin should wear protective gloves during the use.
- Do not use the gel after the “use by” date on the bottle.

**INGREDIENTS:** aqua, paraffinium liquidum, petrolatum, stearic acid, sodium lauryl sulphate, sodium hydroxide, fenoxietanol, ethylhexylglycerin

**PACKAGING**

The product is supplied in the following packagings:  
1000g mini barrel

**DISPOSAL CONSIDERATION**

Product, waste: Chemicals must be disposed of in compliance with the respective national regulations.  
Packaging must be disposed in compliance with the country-specific.

**STORAGE**

Temperature: +10°C ÷ +30°C - Relative humidity: 30% ÷ 75%. Keep container firmly closed, away from direct sunlight and sources of UV radiation

**EXPIRY:** 36 months from production date. Use only 3 month after opening.

**NOTICE:** any adverse event that has occurred in relation to the device should be reported to the manufacturer and any serious incident (MDR Article 2 (65)) should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established;

02.05.2022 V01

	Manufacturer		Temperature limit		Batch-Code		Attention
	User Manual		polyethylene packaging		Use by date		Do not reuse
	no steril		away from UV		Medical device		

**Ultragel Medical Kft.**

✉ 1022 Budapest, Aranka utca 12.  
☎ +36 1 278 3050

✉ [info@ultragelmedical.hu](mailto:info@ultragelmedical.hu)  
🌐 [www.ultragelmedical.hu](http://www.ultragelmedical.hu)