



## INSTRUCTIONS FOR USE

### ULTRASOUND GELS: AQUAULTRA BASIC, AQUAULTRA CLEAR, AQUAULTRA ALOE, AQUA-LUB

ULTRASOUND gels has been designed to be used in ultrasound physiotherapy and diagnostic. The gel for ultrasound guarantees a perfect transmission of the ultrasound waves.

Water soluble It does not contain fatty substances, Easy to remove it doesn't stain, it does not irritate the skin, odourless. It does not contain any active principles that might damage the probes of the echograph equipment. It does not contain salt.

AquaUltra basic is blue, AquaUltra clear is colorless and AquaUltra aloe contains aloe suggested for sensitive skin. Aqua-LUB ultrasound gel has lower viscosity.

#### INSTRUCTIONS FOR USE

- Gel is ready to use.
- Clean and degrease the patient's skin.
- Apply the desired amount of gel.
- Connect and use the equipment and the probes by following the instructions supplied by the manufacturers.
- After the procedure remove all residues of the gel with paper towel or gauze moistened with water.
- Stop using the gel for hypersensitive patients, if skin reactions occur.

#### WARNINGS

- External use only.
- The product is not sterile.
- Do not apply gel on skin affected by abrasions, erythema or wounds.
- Close the bottle after use.
- May cause irritations or reddenings with allergic persons.
- Contact with eyes may cause irritation.
- Analyst with sensitive skin should wear protective gloves during the use.
- Do not use the gel after the "use by" date on the bottle.
- If the package contains a labeled, empty refillable bottle, only the product with the name on the label should be filled

**INGREDIENTS:** aqua, carbomer, sodium hydroxide, EDTA, phenoxyethanol, ethylhexylglycerin (colorless or E 131 or aloe)

#### PACKAGING

Carton with 50 bottles of 260g  
 Carton with 24 bottles, of 500g  
 Carton with 10 bottles of 1000g  
 Carton with 4 cubitainers of 5000g  
 Carton with 4 canisters of 5000g

#### DISPOSAL CONSIDERATION

Product, waste: Chemicals must be disposed of in compliance with the respective national regulations.  
 Packaging must be disposed in compliance with the country-specific.

**STORAGE:** Temperature: +10°C ÷ +30°C - Relative humidity: 30% ÷ 75%. Keep container firmly closed, away from direct sunlight and sources of UV radiation.

**EXPIRY:** 36 months from production date: Use 3 month after opening.

**NOTICE:** any adverse event that has occurred in relation to the device should be reported to the manufacturer and any serious incident (MDR Article 2 (65)) should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established;

02.05.2022 V 01

	Manufacturer		Temperature limit		Batch-Code		Attention
	User Manual		polyethylene packaging		Use by date		Do not reuse
	no sterile		away from UV		Medical device		

#### Ultragel Medical Kft.

✉ 1022 Budapest, Aranka utca 12.  
 ☎ +36 1 278 3050

✉ [info@ultragelmedical.hu](mailto:info@ultragelmedical.hu)  
 🌐 [www.ultragelmedical.hu](http://www.ultragelmedical.hu)